en	Urinary Leg Bags – STERILE
he	שקיות רגל גמבות לרגל – סטריליות
it	Sacche urinarie da gamba – STERILI
de	Urinbeinbeutel – STERIL
fr	Poches urinaires de jambe – STÉRILES
sk	Vrecká na moč na nohu – STERILNÉ

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

©2022 Hollister Incorporated.

912864-122

www.hollister.com/symboldiglossary

EC

REP

Co. Mayo, Ireland

Foxford Road, Ballina

912864

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Draining the leg bag		Tap closed	
Tap open		Tap open	
To remove and dispose (Please follow all local disposal regulations) Remove leg bag straps.		To remove and dispose (Please follow all local disposal regulations) Remove leg bag straps.	
50cm inlet only		50cm inlet only	
Applying the leg bag		Applying the leg bag	
CE mark as required		CE mark as required	
EC Rep or local distributor.		EC Rep or local distributor.	
Notes:		Notes:	
Intended Use:		Intended Use:	
Indications and Description:		Indications and Description:	
Product		Product	
WARNINGS		WARNINGS	
Contents		Contents	
Sterile products		Sterile products	
Non-sterile products		Non-sterile products	
en		en	
INSTRUCTIONS FOR USE		INSTRUCTIONS FOR USE	
Sterile products		Sterile products	
Non-sterile products		Non-sterile products	
Medical Device (as applicable)		Medical Device (as applicable)	
Number of units		Number of units	
Applying the leg bag straps		Applying the leg bag straps	
Precautions		Precautions	
Connecting to a night drainage bag (not included) for extra capacity		Connecting to a night drainage bag (not included) for extra capacity	
To prevent improper drainage do not attach leg bag to knee.		To prevent improper drainage do not attach leg bag to knee.	
Trauma.		Trauma.	
Check the area under the straps regularly for skin irritation or ensure adequate circulation, do not over-tighten.		Check the area under the straps regularly for skin irritation or ensure adequate circulation, do not over-tighten.	
To ensure adequate circulation, do not over-tighten.		To ensure adequate circulation, do not over-tighten.	
To prevent improper drainage do not attach leg bag to knee.		To prevent improper drainage do not attach leg bag to knee.	

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