250 mL 	<span><span>1</span></span> REF <b>9811</b>
<b><span>en</span></b> Male Urinary Pouch	
<b><span>da</span></b> Urindrænagepose til mænd	
<b><span>de</span></b> Urinableiter für Männer	
<b><span>es</span></b> Bolsa para orina masculina	
<b><span>fi</span></b> Miesten virtsapussi	
<b><span>fr</span></b> Poche urinaire pour homme	
<b><span>fr-CA</span></b> Sac collecteur d’urine pour hommes	
<b><span>it</span></b> Sacca urinaria per uomo	
<b><span>nl</span></b> Urineopvangzakje voor mannen	
<b><span>no</span></b> Urinpose for menn	
<b><span>sv</span></b> Urinpåse för män	

www.hollister.com/symbolglossary

(01) 0 0610075 08109 7

**en** Male Urinary Pouch

**Contents:**

- Urinary collection pouch with attached SoftFlex™ skin barrier.

**Product Description**

Urinary collection pouch with a skin barrier suitable for frequent pouch changes or use on fragile skin. The pouch is most suitable for non-ambulatory males with urinary incontinence. It is designed to be emptied and should be attached to a drainage bag.

Made with hydrocolloid skin adhesive and polymer/copolymer plastics.

**Indications and Intended Use:** This product is intended to collect and contain urine for individuals with urinary incontinence.

**WARNINGS**

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

**Precautions**

Discontinue use of this product if any of the following signs or symptoms occur:

- Changes in the skin colour (color) of the penis
- Extreme skin temperature changes
- Swelling, burning, itching, pain, or loss of sensation

**Notes**

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

**INSTRUCTIONS FOR USE**


**To prepare skin and pouch**

Wash genital area with a mild soap and water. Trim the pubic hair. Rinse and dry thoroughly. If desired, wipe the skin that will be in contact with the skin barrier with a skin protective wipe (not included). Please refer to the instructions for use packed with that product. Be sure the drain cap is placed on the pouch drain. If necessary, cut an opening in the skin barrier slightly larger than the penis.


**To apply to skin**

- Remove the protective release paper from the skin barrier.
- Place the penis in the opening of the pouch and press the barrier securely on skin. Avoid wrinkling, which can create channels for leakage. Gently hold the barrier material against the skin so that the user’s body heat will soften the barrier material and enhance its adhesive properties. If desired, apply microporous adhesive strips (not included) around the barrier edges for an added sense of security. Please refer to the instructions for use packed with that product.

- Remove the drain cap at the bottom of the pouch and attach the tubing to a drainage bag.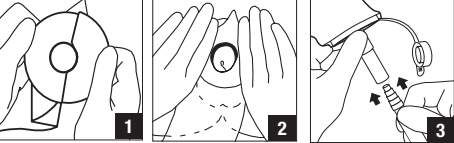

**To dispose.** Please follow all local disposal regulations.

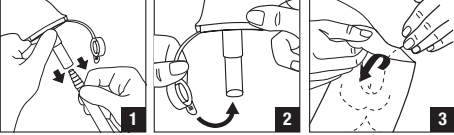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