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INDIKATION

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EN Instruction For Use

INDICATION

Intended for use on patients with a recumbent indwelling bladder catheter, percutaneous renal fistula or urinary sheath for collecting the drained urine by means of a urine drainage system.

The patient must be cared for in a way that is tailored to their individual needs. If the attending physician has given any previous instructions, then there must be observed.

INTENDED USE

It is used for collection of urine drained with a catheter.

KONTRAINDICATION

We are not aware of any absolute contraindications. Restrictions of use may result from the clinical picture of the patient. Leg bag system should not be used on patients with macrohematuria. Our urine drainage systems and their tubes must not come into contact with areas of the skin presenting pathological changes.

WARNING

Please do not use if the package is opened or damaged.

DESCRIPTION

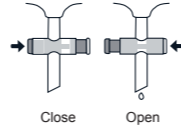
Pediatric range is specifically designed for babies and young children with smaller capacities. Designed to be connected directly to a catheter or sheath for continuous drainage and should only be disconnected when a new bag is required.

Pediatric leg bags can also be connected to a night bag for overnight drainage.

Pediatric leg bag should be changed every 5-7 days.

BEFORE USE

- Wash hands.
- Open peel pouch and remove the leg bag from its original packaging.
- Ensure the t-tap outlet is in the closed position.
- Remove the protective cap from the standard connector.

DIRECT CONNECTION

- Insert the standard inlet connector into the end of indwell catheter fully to allow a secure connection.
- The leg bag should be positioned on the thigh or calf.
- If the leg bag strap is used, then feed the leg bag strap through the centrally positioned eyelets on the bag and ensure the wavy silicone line is facing the leg.
- Make sure the inlet tubing isn't kinked which can restrict the flow of urine.

EMPTYING

- If necessary, remove the drainage tap from the rear flap. Hold it over a suitable collecting container and open the bag by moving the non-return valve in the drainage tap. After the system has been completely emptied, push back the non-return valve, disinfect the drainage tap and insert it into the rear flap for safe and hygienic storage, if available.
- To empty the leg bag using the drainage valve, tilt the valve down 180°.

OVERNIGHT DRAINAGE

If you required additional drainage for overnight use, a night bag can be connected as part of an overnight link system.

- When you go to bed, do not disconnect your leg bag from your catheter or sheath.
- Insert the connector of your night bag to the silicone tube on your leg bag. Insert the connector fully to allow a secure connection.
- Once connected, turn the outlet valve to the open position to allow the overnight drainage
- into the night bag.

- Before disconnecting the night bag in the morning, ensure the leg bag outlet valve is on the closed position.
- Do not reconnect a used night bag.

DISPOSAL

The emptied bag system should be disposed of with the household waste in accordance with national regulations.

CAUTIONS

- Pediatric leg bag should be changed every 5-7 days.
- Do not reuse the pediatric leg bag after disconnection.
- Keep small parts out of the reach of children.

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OBELIS S.A. Bd. Général Wahis 53 1030 Brussels BELGIUM
CMC Medical Devices GmbH Bahnhofstrasse 32 CH-6300 Zug SWITZERLAND