ESPAÑOL

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ENGLISH

PRESENTATION AND DESCRIPTION

URICLAK® is an easy to use transformable external controller for male urinary incontinence. When pressed with fingers inwardly, URICLAK® becomes a circular ring that allows urination without removing it, thus avoiding splashing.

DURATION AND TIME OF USE

The duration of the device is between 3 and 6 months, depending on the use and care of the device by the patient. Its average normal duration is 3 months, and 6 months is the maximum possible duration. After 6 months, it must be replaced for optimal operation and proper hygiene. The patient can use it for an unlimited time while suffering incontinence, but the device must be changed of position as indicated in the section "precautions".

IMPORTANT NOTICES FOR THE CONSERVATION AND USE OF THE DEVICE

- The device should be cleaned with a cotton swab slightly moistened with alcohol. Contact with water should always be avoided. If the above is not fulfilled, the use of life will be reduced well below the 3 months indicated.
- The device cannot be opened more than necessary for placement. Do not open it in a full circle or force it, because the internal structure may break.
- Do not use the cylindrical regulators if it is not really necessary. Most patients do not need them. The use of regulators is only indicated for patients with a special anatomy when the device is too tight, especially when it is new. The objective of the cylindrical regulators is to increase the opening of the device and decrease the pressure that the device exerts on the penis. The excessive use of these cylinders or the incorrect placement of them without following the instructions, can break the device. The cylinders must be inserted turning very slowly and moistened with moisturizing cream.

INDICATION AND PRESCRIPTION

URICLAK ® is indicated for male incontinence of stress or dripping after urination. URICLAK® is for adults only. You do not need a prescription but you should tell your GP/urologist that you are using URICLAK®.

INSTRUCTIONS

URICLAK® is taken with the thumb and forefinger on the ends, with the elongated bulging towards and the two small bulges upward (FIG. 1). Then you press Uriclak® with your fingers laterally inward (FIG. 2), becoming circular, and placed around the penis centering it between the two bumps.

If the pressure on the urethra is too much, enter cylinders tip moistened with moisturizing milk to facilitate its penetration into the holes as shown in FIG 3. **Always start with the smallest diameter. A larger diameter is lower pressure.**

We suggest to urinate every one to two hours to prevent the bladder being completely full, if there are hairs on the skin of the penis, where the device is placed, it can remove them and avoids any hitches. In the case of light urine leakage despite having URICLAK®, you should use a compress level 1 male.

PRECAUTIONS

- It can only be used on dry, healthy and clean skin (without wounds or scars).
- It can only be used for a maximum continuous time of 2 - 3 hours. After that time, you should change its position along the penis, so that it does not press on the same place. It is advisable to reposition it every time you urinate (every 2 - 3 hours)
- In case of discomfort or redness, it is advisable to change the position of the device (up or down), to adjust the pressure with the regulators or to use a moisturizing milk on the skin. If the discomfort or redness does not go away, stop using it for a while until the discomfort goes away.
- Do not use if the soft coating is deteriorated.
- Do not use if the internal metal structure is broken.
- You should stop using if you experience any inflammation.
- Can only be used by patients with full cognitive capacity and body autonomy. This device is not recommended for use on severely handicapped patients except that the patient is under the strict direct supervision of another person.
- The use of this device is not recommended if the patient suffers from incontinence due to internal obstruction

POSSIBLE SIDE EFFECTS

Sometimes redness or inflammation may appear on the skin and if it is not resolved by changing position, using the pressure regulators or spreading moisturizing milk on the skin, stop using it for as long as necessary, until these effects disappear.

LEGAL NOTICE

URICLAK® has an international patent and is a registered trademark. All rights reserved. Class I medical device in accordance with European regulation 745/2017 and ISO 13485.

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