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EN	INSTRUCTION FOR USE - Catheter fixing dressing
Product name:	Adhesive Wound Dressing
Trade name:	Catheter fixing dressing
Material:	Made of medical polyurethane film composite non-woven fabric, medical pressure-sensitive adhesive, buckle tape, release paper.
Intended purpose:	Adhesive Wound Dressing is used for various wound care such as exudate absorption, cleaning care, fixation of catheter.
Intended users:	The product shall be used by or under the supervision of a healthcare professional.
The combination of the device with product:	Not applicable
Contraindications:	People who are allergic to product materials.
Complications or side-effects:	Unknown
Instruction of use:	Before using this product, clean and disinfect the wound, tear off the small package, then tear off the protective layer at both ends of the back, stick the adhesive surface to the skin, and then fix the catheter on the catheter holder.
Warnings and precautions:	1) This product is a disposable product 2) If the product is expired, it cannot be used 3) It is strictly prohibited to use if the inner package is damaged 4) Disinfect the wound before use, and do not touch the protective pad with your hands after unpacking. 5) The sticky time of the product is long, please change it in time to keep the wound clean 6) If skin irritation occurs during use, please stop using.
Storage:	Store in rooms with a stable temperature from 0 °C - 45 °C and humidity from 10% - 70%. Keep away from direct sunlight and keep dry. Store in a non-corrosive gas environment, away from light, moisture, fire and flammable materials.

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Hubei Qianjiang Kingphar Medical Material Co., Ltd. Yuanguang Road 433100 Qianjiang PEOPLE'S REPUBLIC OF CHINA
Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80 20537 Hamburg Germany
CMC Medical Devices GmbH Bahnhofstrasse 32 CH-6300 Zug Switzerland